

TEST KIT

- 32 ready to use Reagent Caps and Cuvettes
- Internal calibration stored on RFID card
- 4 month shelf-life
- For use with MyCare Insite Analyser and Touch Screen
- For use with the 20 μ L (neutral), white plunger, Sarstedt Minivette[®] POCT Capillary¹

MEASUREMENT

Sample: capillary whole blood
Stored calibration: 0 – 1390 ng/mL
Results in less than 7 minutes
Standardized to serum results

RECOVERY & LINEARITY

Recovery: 83% to 104%
Linearity: $\leq 15\%$ deviation across measuring range of 170 to 1390 ng/mL

¹Not included in test kit, sold separately

SPECIFICITY

Detects clozapine

No interference from:

1. Related compounds:

- Norclozapine
- Clozapine N-oxide
- Quetiapine
- Olanzapine

2. Endogenous Interferents:

- Rheumatoid Factor
- Total Protein
- Icterus
- Lipemia
- Hemolysis
- Hematocrit
- Human Serum Albumin
- Human IgG

3. Co-administered Medications & Cross-reactants, Including:

- 2nd & 1st Generation Antipsychotics
- Antidepressants
- Corticosteroids
- Opiates
- Over the Counter Drugs


The MyCare Insite is designed to give results where and when it's needed most: at the point-of-care.

The MyCare Insite delivers blood levels in less than 7 minutes from a finger stick. The assay is standardized to give clozapine serum values equivalent to reference methods.

Clozapine Test




LINEARITY

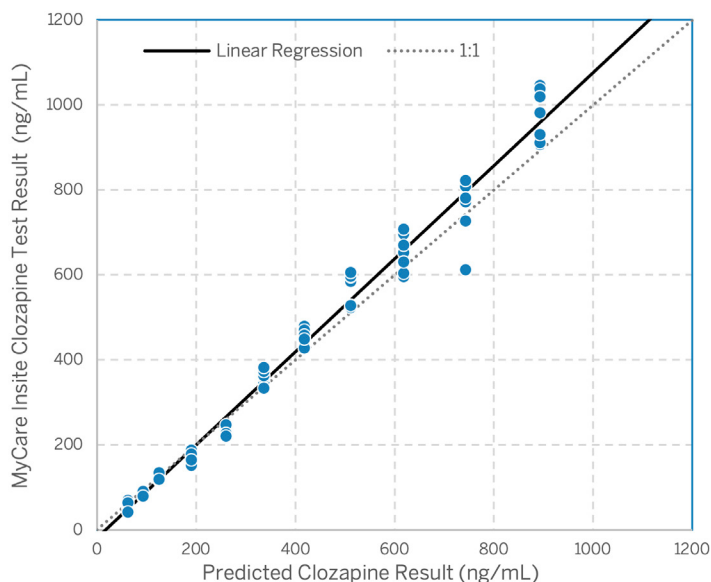


Linearity demonstrated from 170 - 1390 ng/mL.

CLINICAL ACCURACY

Agreement between MyCare Insite and reference method. This shows a comparison with 304 patient samples.

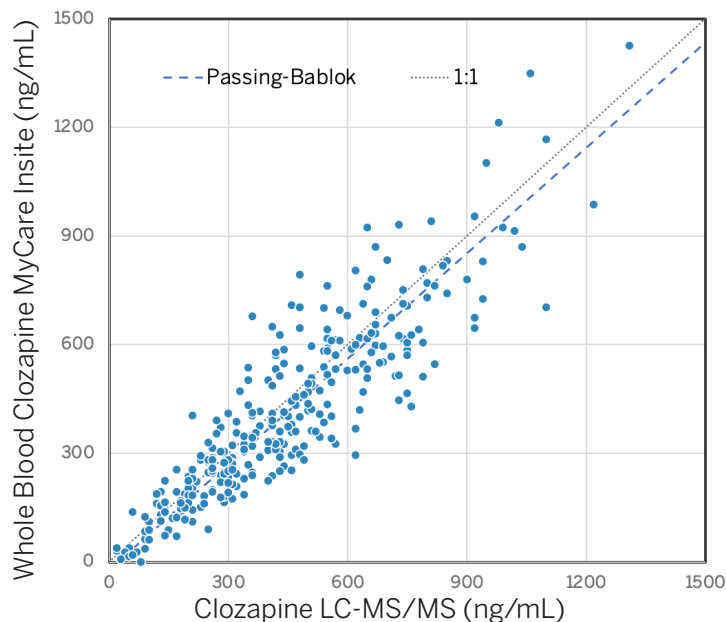

PRECISION

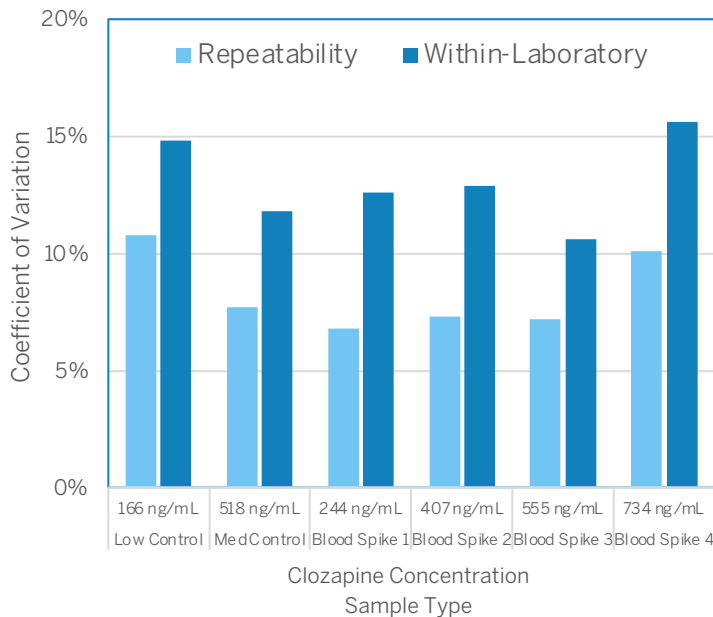

Precision with multiple operators (>5), and multiple Insites (>20), gave repeatability of <11% and within-laboratory precision of <16% for whole blood samples.

CONTACT SALADAX

 **Email:**
hello@saladax.com

 **Call:**
+1 (610) 419-6731

 **Visit:**
mycaretests.com
saladax.com